

# firstMED



Model: **ECG-300**

ELECTROCARDIOGRAPH

EKG CİHAZI

ELECTROCARDIOGRAPHIE

ЭЛЕКТРОКАРДИОГРАФ

AMÈRA EKG'YÊ















جهاز تخطيط القلب

اللكتروكارديوغرافي

**EN** Instruction Manual

[www.firstmed-health.com](http://www.firstmed-health.com)

# Symbols

	CE mark		Keep the carton straight.
	Read the instructions carefully before using this device.		Fragile, handle with care.
	Waste Electrical and Electronic Equipment Directive		Storage temperature limit.
	Manufacturer		Type CF equipment
	Caution-Consult accompanying documents.		Class I equipment
	Keep away from sunlight.		Manufacturing date
	Keep dry.		Serial number

# Instructions to User

Dear User, thank you very much for purchasing our product.

This Manual is written and compiled in accordance with the council directive 93/42/EEC for medical devices and harmonized standards. The Manual is written for the current ECG-300 Electrocardiograph. In case of modifications and software upgrades, the information contained in this document is subject to change without notice.

The Manual describes, in accordance with the ECG-300 Electrocardiograph's features and requirements, main structure, functions, specifications, correct methods for transportation, installation, usage, operation, repair, maintenance and storage, etc. as well as the safety procedures to protect both the user and equipment. Refer to the respective chapters for details.

Please read the Manual very carefully before using this equipment. These instructions describe the operating procedures to be followed strictly, Failure to follow these instructions can cause measuring abnormality, equipment damage and personal injury. The manufacturer is NOT responsible for the safety, reliability and performance issues and any monitoring abnormality, personal injury and equipment damage due to user's negligence of the operation instructions. The manufacturer's warranty service does not cover such faults.

## **WARNING:**

- **All the electrodes connected to the patients directly must be correct and tried.**
- **Make sure the battery voltage is normal when choosing the UPS**
- **The AC power cable and the patient cable could not be enlaced.**
- **Please peruse the relative content about the clinical restrictions and caution.**
- **This device is not intended for treatment.**

# Contents

1	Main Technical Specifications.....	3
2	Security Notice.....	3
3	Device Characteristics .....	5
4	ECG-300 View .....	6
5	Operation Direction .....	8
6	Preparation Before Operation .....	8
7	Attention Points of during Operation .....	9
8	Paper Loading .....	10
9	Electrode Placement .....	10
10	Grounding and Power Connection .....	12
11	Battery Operation Direction .....	12
12	Control Panel and Keys .....	13
13	Troubleshooting.....	26
14	Maintenance and Storage.....	29

## 1 Main Technical Specifications

### 1.1 Normal work environment

#### Operation

- a) Environment temperature: +5 °C~+35 °C
- b) Relative humidity: ≤80%
- c) Power supply: AC:220V±22V, 50Hz±1Hz  
DC: 7.4V, 3700 mAh rechargeable lithium battery
- d) Atmospheric pressure: 86kPa~106kPa

#### Store and Transportation

- a) Environment temperature: -10 °C~55 °C
- b) Relative humidity: ≤95%
- c) Atmospheric pressure: 50kPa~106kPa

### 1.2 Input way: Floating and defibrillation protection

### 1.3 Lead: Standard 12 leads

### 1.4 Patient leak current: <10μA

### 1.5 Input impedance: ≥50MΩ

### 1.6 Frequency response: 0.05Hz~150Hz (-3dB)

### 1.7 Time constant: Time constant>3.2s

### 1.8 CMRR: >60dB, >100dB( Add filter)

### 1.9 EMG filter: 35Hz(-3dB)

### 1.10 Recording way: Thermal printing system

### 1.11 Specification of recording paper: 80mm (W)×20m(L) High-speed thermal paper

### 1.12 Paper speed: 25mm/s、50mm/s, error:±5%

### 1.13 Sensitivity choice: 5、10、20mm/mV, error:±5%。Standard sensitivity is 10mm/mV±0.2mm/mV

### 1.14 Auto-record: record following the auto-mode, auto leads-changing, auto measurement and analyse.

### 1.15 Manual record: record following manual mode, manual leads-changing.

### 1.16 Measurement parameters: HR, P-R interval, P Duration, QRS Duration, T Duration, Q-T interval, Q-Tc, P Axis, QRS Axis, T Axis, R(V5), S(V1), R(V5)+S(V1)

### 1.17 Product safety type: Class I, Type CF, there is defibrillation and pacing protection circuit.

### 1.18 Enduring polarization voltage: ±300mV

### 1.19 Noise level: ≤15μVp-p

### 1.20 Fuse Specification: 2 pcs φ5×20mm AC time lag; T250mA/250V(Power Supply:220V)

### 1.21 Size: 315mm(L)×215mm(W)×77mm(H)

### 1.22 Net Weight: 2.25Kg

## 2 Security Notice

2.1 The power supply should be grounded properly before operation.

2.2 If the ground cable is not integrated, the device must be run with built-in power supply.

2.3 Please pull out power supply plug before changing the fuse.

2.4 This device must be operated and preserved by professional personnel.

2.5 The operator must read this user manual carefully before operation, and operate the device

according to operation regulation strictly.

2.6 The design of this device has mature consideration of security, but operator should never neglect attention to device state and patient's situation.

2.7 Please turn off the instrument and pull out power supply plug before clean and disinfection .

2.8 Please don't operate this device in environment contain flammable anaesthesia gas.

2.9 If this device is used with cardiac defibrillator or other electric stimulate devices at same time, please choose Ag/AgCl chloride chest electrode and ECG lead with prevent-defibrillation function. To prevent the metal electrode burn patients' skin, the disposable chest electrode should be used if the defibrillation time is over 55 seconds. It is better that do not use this device with other electric stimulate device at same time. If must be used at the same time, there must be professional technician guide on the scene.

2.10 When other devices are connected with this ECG instrument, they must be Type I devices which accord with IEC60601-1. Because the total amount of leakage current may hurt patients, the monitoring of leakage current is carried out and taken charge by connect devices.

2.11 Following descriptions concern special attentions in ECG measurement and interpretation.

- (1) P wave and Q wave are not always reliable in the archive of intensive muscle artifact or AC interference. So are the ST segment and T wave.
- (2) Winding and unclear ends of S wave and T wave may lead to tolerance in measurement.
- (3) In archive R wave is left out due to the low voltage of QRS wave or any leads falling off, the measured heart rate may deviate greatly from the correct one.
- (4) Axis calculation and identify the QRS borderline are not always reliable in the archive of the low voltage of QRS wave.
- (5) Occasionally, frequent ventricular premature complexes may be identified as dominant beat.
- (6) Merging of versatile arrhythmia may result in untrustworthy measurement because of the difficulty in distinguishing P wave in such situation.
- (7) ECG-300 is designed to carry on ECG trace interpretation immediately after the measurement. It is this interpretation that does not give report on all possible heart problems and may sometimes not comply with the doctor's diagnosis. Therefore, the final conclusion concerning each patient is up to the doctor basing on patient symptom, the unit ECG-300 's interpretation and other examination.

### 3 Device Characteristics

3.1 Recording system: Thermal-array (8 dots/mm), you should not adjust anything Frequency Response is up to 150Hz.

3.2 The device can record real time clear and exact three channel ECG waveform and remark continually. The remark includes: lead sign, sensitivity, paper speed, filter state, etc.

3.3 Under automatic mode, just press the button once, it starts record procedure, which can enhance your work efficiency.

3.4 Soft keyboard control, more convenient for operation. TFT screen shows the working status, more clear for observation.

3.5 Safety Class: Class I, Type CF.

3.6 The power supply includes both AC/DC. This device includes built-in lithium rechargeable battery,

3.7 This instrument can record 150 pieces of ECG waveform and print 90 minutes continually under the best DC state.

3.8 This instrument can store more than 1000 pieces patient's data, more convenient for data review and statistic.

3.9 The figure of whole device is elegant and gliding.

3.10 According to dependence degree of deleterious fluid, this device belongs to common device.




3.11 According to the safe degree used under the condition with flammable anesthesia gas mixed with air ( or oxygen, nitrous oxide), this device belongs to the device which can't be used under the condition with flammable anesthesia gas mixed with air( or oxygen, nitrous oxide) .

3.12 Digital signal processor for effective inhibition of baseline drift, interference, and the like.

3.13 The instrument has function with regular auto-measurement of ECG waveform parameter, auto-analyze and auto-diagnostic, it will help to reduce doctor's abundance and improve working effect.

3.14 According the working mode class, this device belongs to non-continuous working device.

3.15 Explanation of some symbols in this device:

~AC	AC work mode
OFF	Power supply is disconnected
ON	Power supply is connected
	Equipotential point
	Places need to be noticed, please refer to user manual
	Device type is CF, which has defibrillation protection function



RS232 connector



USB connector




PATIENT Lead connector

#### 4 ECG-300 View

##### 4.1 Device View




Front view

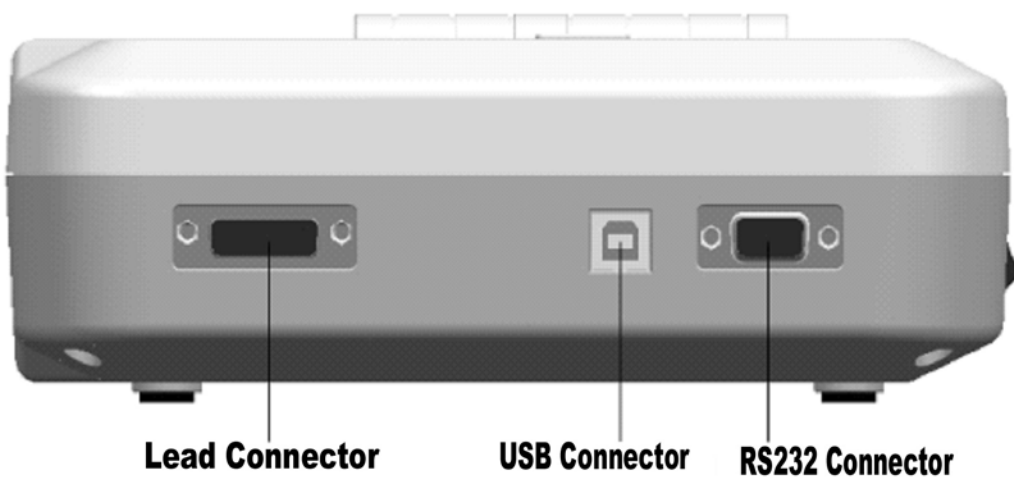

Side view



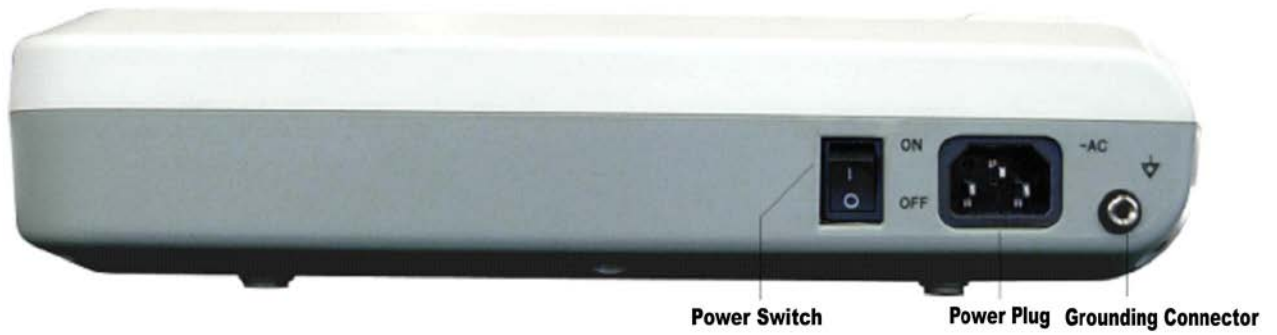

Rear view



Bottom view

#### 4.2 Key Definition



Function key: on/off



Function key: plus adjust



Function key: paper speed adjust



Function key: filter function select



Function key: menu



Function key: switch printing mode



Function key: 1mV marker



Function key: print



Function key: enter



Function key: upwards



Function key: downwards



Function key: leftwards



Function key: rightwards

#### 4.3 Indicator Definition



The indicator turns green when there is AC power supply, and when the indicator turns green and red same time it is being recharged.



Indicator for instrument when power on.

## 5 Operation Direction

5.1 You are required to read this operation manual carefully before operating so as to ensure taking safe and effective operation of the instrument.

5.2 Installation and maintenance of the instrument shall be carried out as the following

5.2.1 There should be no high voltage cable, X radial instrument, ultrasound instrument and electrotherapeutics instrument, etc around the ECG instrument.

5.2.2 Do not use or reserve the instrument in the place where the air pressure is too high, temperature and humidity are over the common standard, the ventilation is not good, dust is too much, there is gas containing salt and alkali and chemical medicine.

5.3 The instrument should be put on flat place. Take and put it lightly when move it. Avoid too strong vibration and shock.

5.4 AC frequency and voltage value should accord with requirement, and has enough current capacity.

5.5 Please put the device at the place where is easy to be grounded. Do not connect the patients and the patients connecting cables with other conductors including ground or beds which can be conducted well with ground.

## 6 Preparation before Operation

6.1 Check that the instrument properly grounded and that cable connections safe or not.

- 6.2 Make sure all electrodes directly connected with patient is properly and firm.
- 6.3 Check the output voltage when choose the DC UPS.
- 6.4 Smear the gel separately, avoiding the short circuit caused by the chest electrode touch one another.
- 6.5 AC power cable cannot be enlaced with ECG cable.

## 7 Attention Points of during Operation

- 7.1 Pay attention to the patient and instrument condition constantly.
- 7.2 Patient and instrument can only be connected ECG cables.
- 7.3 Keep close observation of the patient and instrument, to make sure they are not moved during operation.
- 7.4 Turn off the instrument after using.
- 7.5 Turn off the power, and remove the ECG cables slightly without force.
- 7.6 Properly keep the instrument and spare parts for operation next time.
- 7.7 Paper Loading




- 7.7.1 Dimension of the high-speed thermal Recording paper used in this instrument is: 80mm (W)×20m(L)

7.7.2 Open the cover of paper cabinet, take out the paper axis and install recording paper according to the figure into the proper position inside.

7.7.3 Close the cover of paper cabinet. It's recommended to leave 2 cm of recording paper outside.

## 8 Paper Loading

8.1 Message “No Paper.” will be displayed on the LCD whenever recording paper is run out.

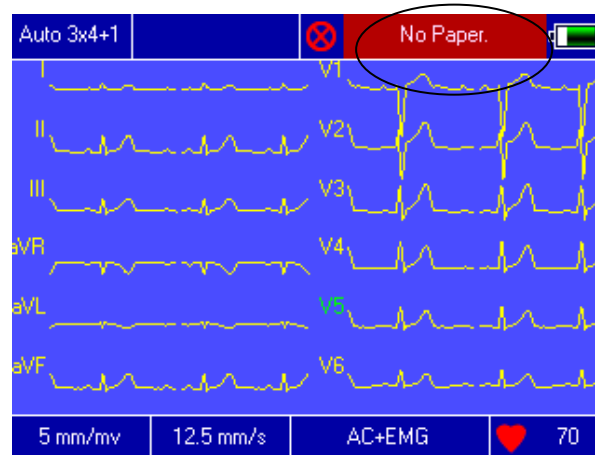

8.2 Specified paper of high sensitivity is recommended for high-quality prints. Other kind of paper may not render a clearly permanent trace and may damage the printing mechanism. Please consult distributor or manufacture for detail of how to purchase the paper.

8.3 Failure of the recording paper might be affected by high temperature, bad humidity or direct sunlight. For long storage, the recording paper should be placed in dry, dark and cool area.

8.4 Substance may caused stain of the recording paper:

Gel, glue, and wet diazo compound paper including their organic solvent.

8.5 Substance may caused the waves fade away:

File folders made of soft PVC material, plastic etc; eraser and magnetic tape contains plasticizer; fluorescence, and stamp-pad ink.

## 9 Electrode Placement

Advice: Set the chest electrode first, then the limb electrode.

9.1 Chest Electrode( See Figure 4)

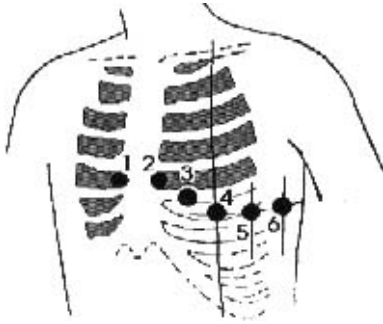


Figure 4

Attach the chest electrodes to the locations as following:

V1: Fourth inter-costal space at right border of sternum.

V2: Fourth inter-costal space at left border of sternum.

V3: Midway between V2 and V4.

V4: Fifth inter-costal space at left mid-clavicular line.

V5: Left anterior axillary line at the horizontal level of V4.


V6: Left mid-axillary line at the horizontal level of V4.

Clean the skin where chest electrodes are to be attached with alcohol, then apply ECG cream to here around 25mm in diameter and to the edge of chest electrodes, and press and attach the electrodes to the positions from V1-V6.

Keep in mind that the electrodes' coming into contact with each other or cream's overlap from one position to another is not allowed.

### 9.2 Limb Electrode

Electrodes should be placed on the soft skin of hands and feet. Clean all the limb electrodes and the positions around to which limb electrodes are to be attached with alcohol before applying ECG cream to them, then firmly attach the electrodes to the positions

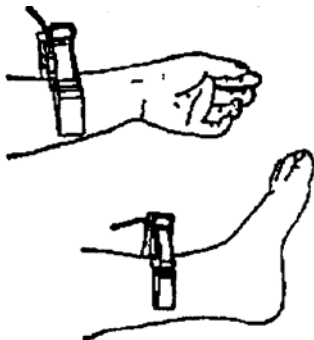

Caution: Screw tightly the knob of ECG cable's plug after it inserted to the instrument.

### 9.3 Check-List for Electrode connection and ECG cable

Electrode Location	Electrode Code	Socket Number
Right Alarm	RA/R	9
Left Alarm	LA/L	10
Left Leg	LL/F	11
Right Leg	RL/N	14
Chest 1	V1/C1	12
Chest 2	V2/C2	1
Chest 3	V3/C3	2
Chest 4	V4/C4	3
Chest 5	V5/C5	4
Chest 6	V6/C6	5

## 10 Grounding and Power Connection of Instrument

Make sure the status of the instrument is power off, and then make the instrument be properly grounded through a 3-prong outlet. When the outlet, a grounding cable may be utilized to connect the grounding terminal of the instrument. Do not use other pipeline. Properly grounding could guarantee the safety and prevent from the interference of AC power and electromagnetic wave.







## 11 Battery Operation Direction

11.1 This instrument designed with a the built-in sealed maintenance-free rechargeable lithium battery, and has automatic charge and discharge monitoring system. The instrument recharges the battery automatically when connect to AC power supply. The LCD screen will show the current power state at the top right corner when the instrument turns on (see 12.4). It needs about 4 hours for battery charge after discharge absolutely.

11.2 The device can continuously print 90 minutes and work 4 hours without printing after the battery fully charged. When it working, the LCD screen displays the signal of the battery status in 5 degree. When the power of battery is too low to operate, the instrument will turn off automatically to avoid damage to the battery.

11.3 The battery should be recharged in time after exhausted using. For long storage, the battery is to be recharged every 3 months. The battery life can be extended by doing so.

11.4 Seven status of the battery power displayed on LCD as following:

No.	mark	description
a	...	Unknown status, normally displayed while the instrument being turned on within 1 minute
b		Using AC power
c		Using battery, and full power
d		Using battery, volume : 3/4
e		Using battery, volume: 1/2
f		Using battery, volume : 1/4
g		Using battery, but lower power, suggest to recharge the battery or use AC power supply

Note: When charging, the battery icon shift from f to c.

11.5 When the battery can not be recharged or works no more than 10 minutes after fully charged, please change the battery.


Attention!!!

- Do not directly connect both “+” and “-” polars of battery with wire, otherwise it might cause fire hazard.
- Possible explosion hazard if it kept nearby the ablaze area.
- You should not open or disassemble the battery .

## 12 Control Panel and Keys

### 12.1 Main Interface

Show as following:

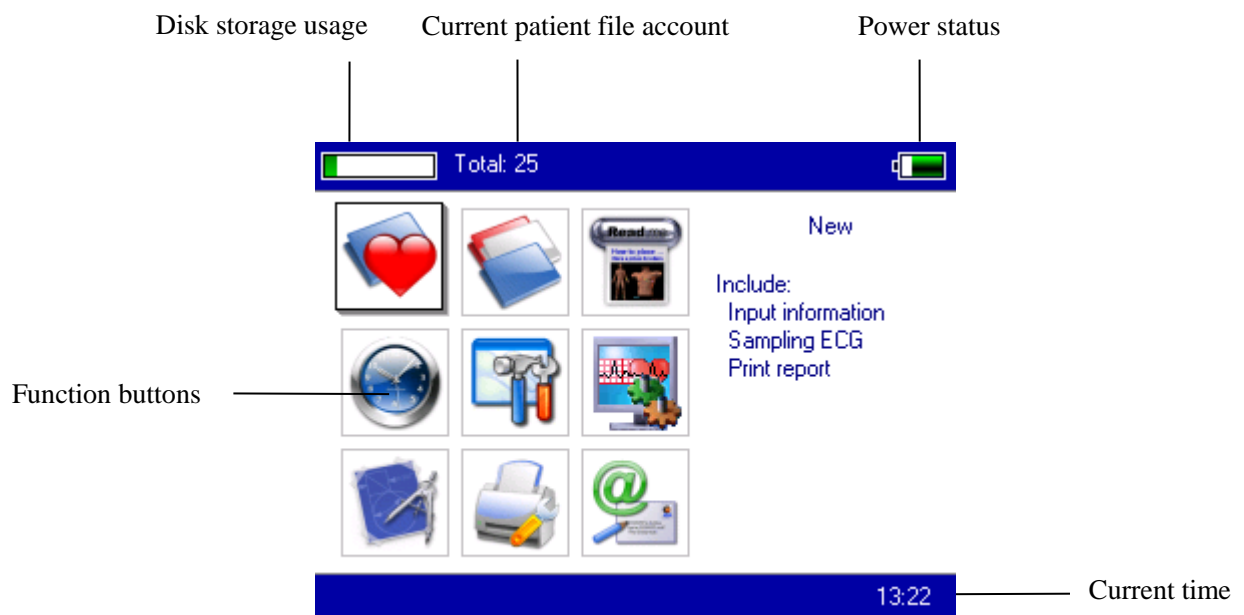

Power status: Please refer to 12.4

Keypad:



Enter sampling interface. when the instrument is powered on, it will automatically start this operation.



Enter Archive management interface, query , modify or delete archive information



See sketch map for electrodes placement



Date and time settings



System settings



Sampling settings



Analysis parameters settings. settings for each parameter using for automatic analysis




Printing settings, set printing mode, style and content.



About us, display information about our company and software version

## 12.2 Sampling Interface




Select  on the main menu or choose shortcut key  to enter sampling interface.


Attention: Because of the “setting”, Patient information maybe input before sampling signal, rest with the option : inputting archive information.



It displays multiform lead waves, including 3 Leads per screen, 6 Leads per screen, 12 Leads per screen.

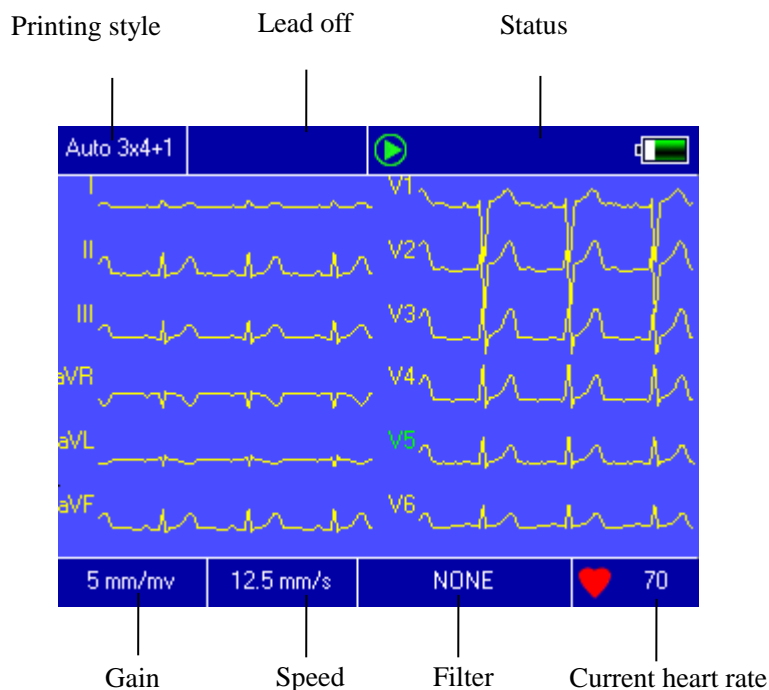

Sampling interface of 12 leads style can be displayed as following:







Stop Sampling: Press key  on the keypad to stop sampling and return to main interface.


Lead Change: If the show style is 3 leads per screen or 6 leads per screen, you can press key  /  to show the other leads.

Switch Lead Style: Press key  /  to switch the lead style among 3 leads per screen, 6 leads per screen and 12 leads per screen.

Lead Off: Under Demo mode, it displays "DEMO ECG". Under sampling mode, it displays information of lead off.

Change Recording Style: Press this key  to change recording mode among auto 3×4 +1, auto 3×4, auto 2×6 +1, auto 2×6, auto 4×3, rhythm 4, rhythm 3, rhythm 2 and manual mode.

Adjust Gain (Sensitivity Selection): Sensitivity is to be selected by pressing key . User can choose 5mm/mV, 10mm/mV or 20mm/mV.

Adjust the speed: Use the key of speed adjusting  to change the speed :25mm/s or 50mm/s.

Shift the filter: Use the key of filter selection  to shift between non-filter, AC, EMG, DFT, AC+EMG, AC+DFT, EMG+DFT, AC+EMG+DFT.

Display the calibration: Use the key of "1 mV"  to display the marker of 1 mV on the screen.

Print/ finish the print: Use the key of "print"  on the keyboard, then can start or stop a printing operation.


Automatic mode: After starting printing, the system will print and store synchronic twelve leads waveform automatically. The length decided by the setting item in the printing option. And also according to the setting item, print out the data and result which analysed automatically and the system can finish the printing automatically.

Manual mode: After starting printing, the user should print out the real waveform by shifting the group of leads. That means the ECG waveform printing under the manual mode is non-synchronic, and can not be saved. And the user should end the printing by press the key again.

During printing, the printing state includes:

Display content	Remark
Process...	in the process of printing
Waiting...	in the process of ending printing
No Paper.	Lack of paper, the user should restart the operation after filling papers.
Print Timeout.	The connection between system and printing sub-system broke.
ECG Timeout	The connection between system and sampling sub-system broke.
Low Power	Low power, the system can not start the print work.

On this interface, press the button **SET** on key board, the system will enter the shortcut setting interface:

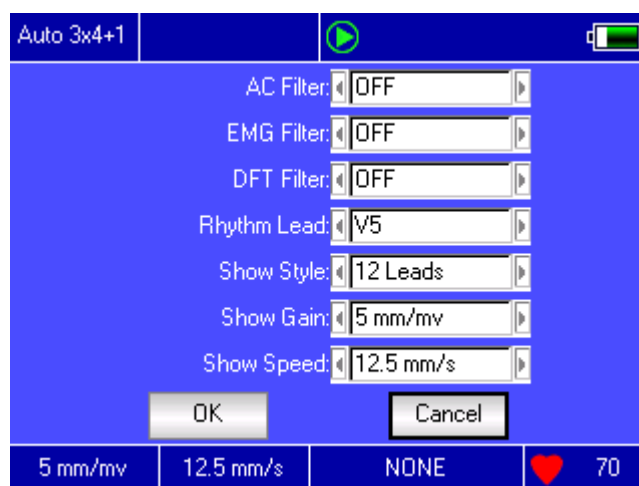

Select **【OK】** button, the system will apply new settings and return to sampling interface. Select **【Cancel】**, the system will return to sampling interface without apply the new settings.

The each function of option is shown in the following table.

item	Optional content	remark
AC Filter	[ON]/[OFF]	Setting of using AC Filter or not


EMG Filter	[ON]/[OFF]	Setting of using EMG Filter or not
DFT Filter	[ON]/[OFF]	Setting of using DFT Filter or not
Rhythm Lead	Any one of 12 leads	Setting the rhythm lead to print ECG in rhythm print mode.
Show Style	[3 Leads]/[6Leads]/[12Leads]	Setting of wave show style.
Show Gain	[5mm/mV]/[10mm/mV]/[20mm/mV]	Setting of wave show gain.
Show Speed	[12.5mm/s]/[25mm/s]/[50mm/s]	Setting of wave show speed.

### 12.3 Inputing Archive Information

According to the different setting items(refer to 13.8),user can input the patient archive before or after sampling, and also can input blank archive.The input box as following:

Choose any input-box, as pressing **SET** key, the “soft keyboard” will pop out as following.The function of **【Caps】** button on “soft keyboard” is to change the number key and lowerarchive to punctuation and upperarchive. Press **【OK】** will confirm input and exit this interface.

There maybe a limit of character according to the content input. And then the limited character will be gray and unavailable, as following:

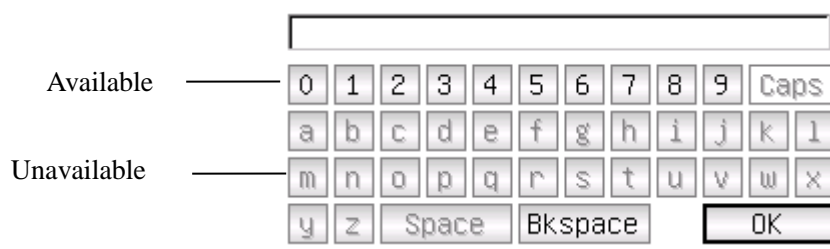

## 12.4 History Archive Management

In the main interface, select the button of , then you can enter the archive management interface, as following.





Current archive sum / Total archive

Total: 22		Current: 6 / 22	
Date and time	ID	Name	Sex
2008-06-11 08:30:53	0001	GaoXiaohua	F
2008-06-10 15:31:15	0002	SongGuang	M
2008-06-10 15:12:39	0003	LiCui	F
2008-06-10 14:51:26	0004		
2008-06-10 14:27:21	0005		
2008-06-10 14:25:03	0006	ZhangXiaohua	M
2008-06-04 14:08:49	0007		
2008-06-04 11:57:12	0008	QinXiaohua	F
2008-06-04 09:33:02	0009		
2008-06-04 09:14:47	0010	CaiLinlin	M

Adv-opr k << >> >| Review Delete Return


Current selected archive ———> (row 6)  
 Archive list ———> (table)  
 Display advanced menu ———> (Adv-opr)  
 Pages roll buttons ———> (k, <<, >>)  
 Delete the selected archive ———> (Delete)  
 Exit this menu ———> (Return)  
 Display review dialogue box ———> (Review)

This interface shows all the storage archive. The users can use the searching function (refer to 12.5 archive querying) to select the required archive; and edit any archive by modifying or deleting operation; besides the user can review the saved archive information. (refer to 13.6 archive review).

-  : Go to the first page of archive list.
-  : Go to the last page of archive list.
-  : Go to the previous page of archive list.
-  : Go to the next page of archive list.


## 12.5 Archive Querying.

Choose **【Adv-Opr】** in the archive management ,then it shows as following :

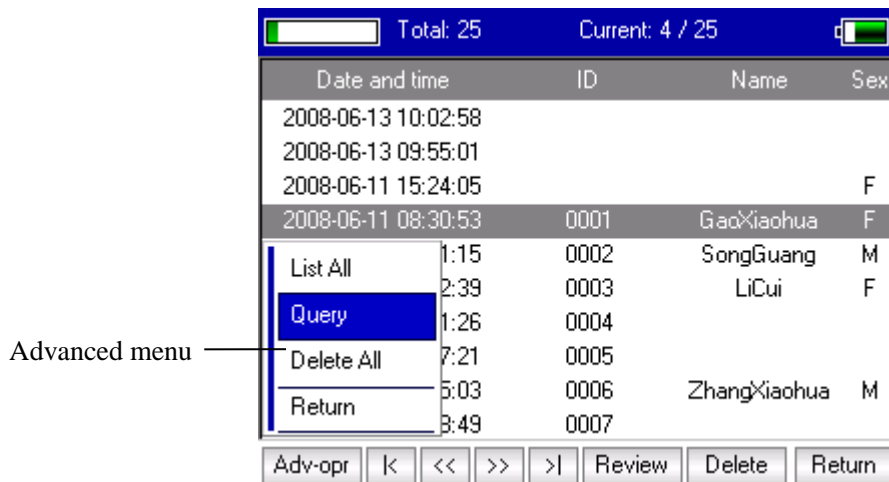

Select **【Query】** can start a archive Query dialogue box as following.

Input searching condition, and select **【Select】** button, and the user can get expected result. The function of **【Clear】** is to clear the query condition input.

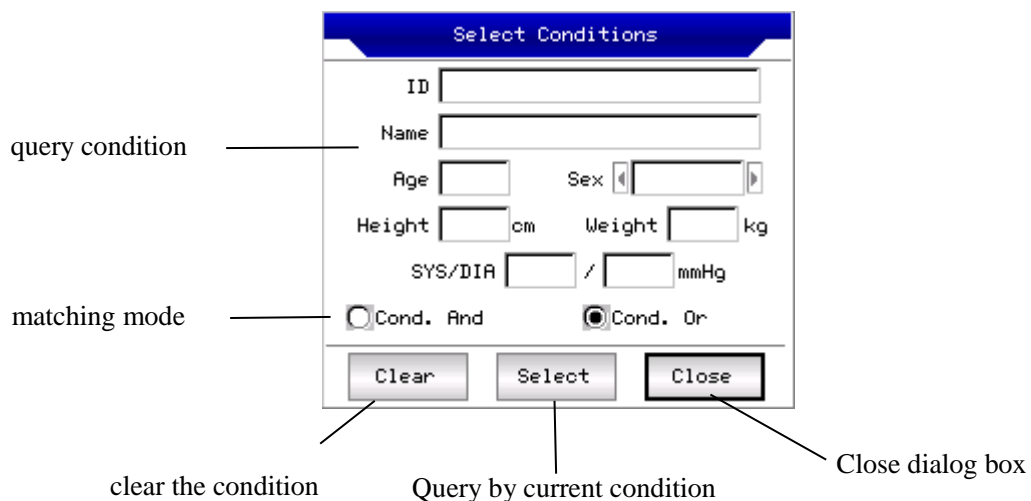


**【Cond. and】** and **【Cond. or】** indicate the matching mode of searching condition, the user can select either. If select **【Cond.and】**, the searching result will fit all the conditions input simultaneity; if select **【Cond.or】**, the searching result will display the ones which fit any of the conditions.


Suggestion: On the conditions of large number of patient archive, should input the confirmed searching conditions, select **【Cond. and】**, can find out the certain patient archive immediately.




## 12.6 Archive Review

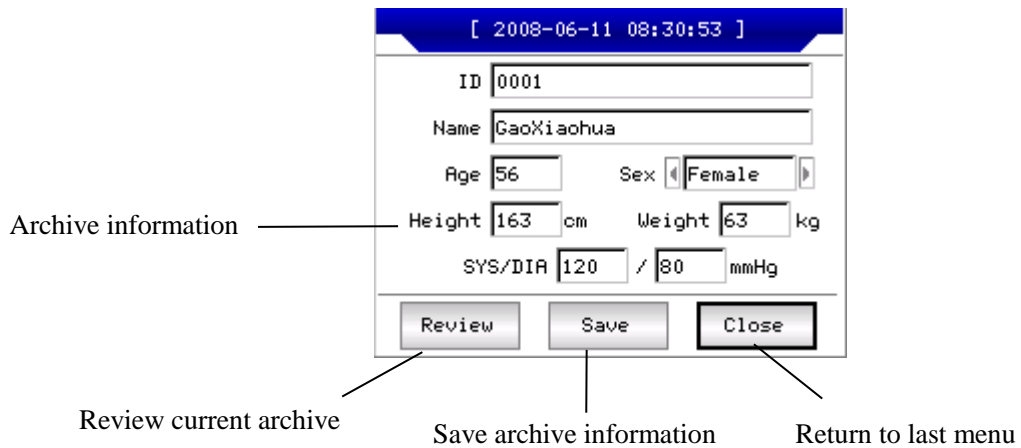
On Archive management interface, after moving focus on the right patient archive being reviewed, select **【Review】** can start the following dialogue box which shows the patients archive information, users can modify here, select **【Save】**, the change, which is not reversible, will be saved.

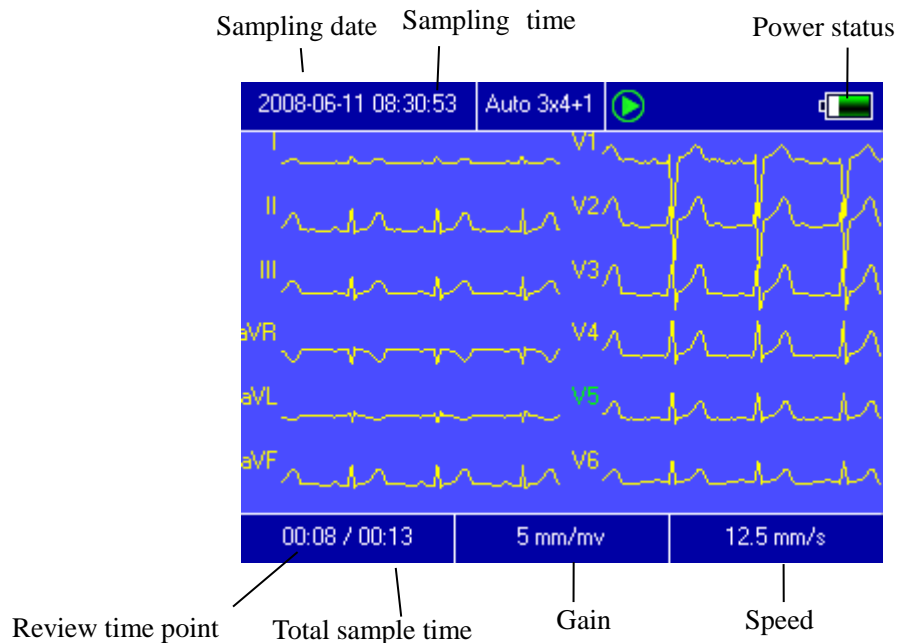

Make sure the correct selection, select **【Review】** button, and can go into the following review interface which is similar to sampling interface.



In this interface, the user can adjust the time segment of the display waveform by  and , and also the speed and gain.( refer to 13.2 sampling interface).In this state , user can print by pressing  . pressing **SET** , system will enter the shortcut settings interfaces following:

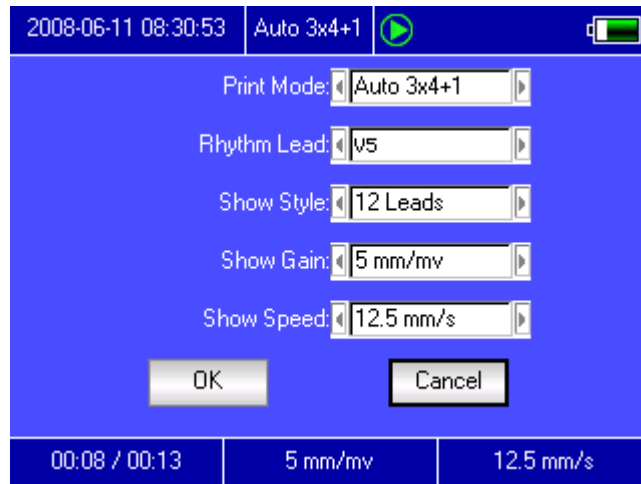


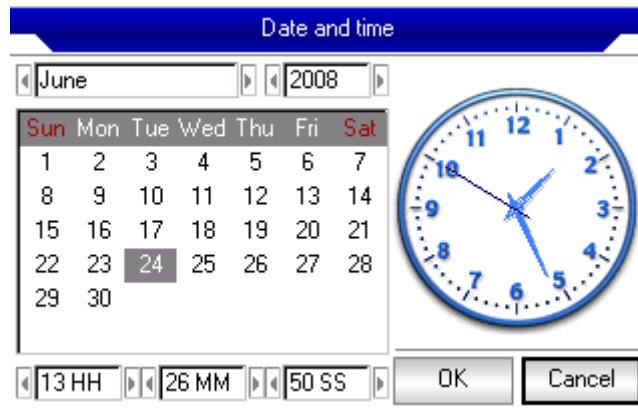
Select **【OK】** button, the system will apply the new settings and return to review interface. Select **【Cancel】** button, the system will return to review interface without applying the new settings.





Each function of the option is as following:

item	Optional content	remark
Print Mode	[Auto 3×4]/[Auto 2×6]/[Auto 4×3] and any mode suit current archive	Setting of printing mode.
Rhythm Lead	Anyone of 12 leads	Setting the rhythm lead to print ECG in rhythm print mode.
Show Style	[3 Leads]/[6 Leads]/[12 Leads]	Setting of wave show style.
Show Gain	[5mm/mV]/[10mm/mV]/[20mm/mV]	Setting of wave show gain.
Show Speed	[12.5mm/s]/[25mm/s]/[50mm/s]	Setting of wave show speed.


## 12.7 Date and time Settings


In the main interface, select  button, can start the following dialogue box showing the date and time settings.



In this interface, the users can select  and  key to shift to all the items, using  and  to adjust the options content.

## 12.8 System Settings

In the main interface, select  button, can start the following system settings dialogue box.

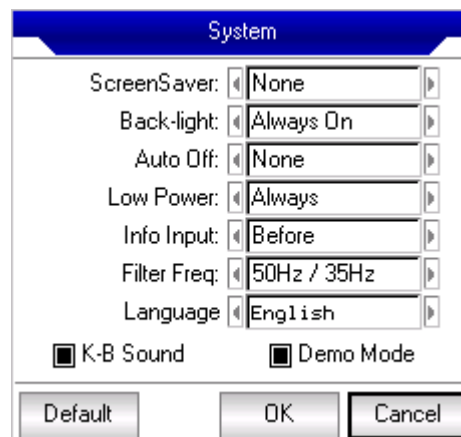

In this, select the button **Default**, the system settings will back to the default.


The each function of option is as following:


item	Optional content	remark
Screen saver	None/30Seconds/1Minute/2Minutes/5Minutes/10Minutes	The screen saver will be active after the selected time. "None" means that this function will not be used.
Back-Light	30Seconds/1Minute/2Minutes/5Minutes/10Minutes/Always On	The back-light will be turned off after the selected time. "Always On" means that the back-light will be turned off never.
Auto off	1Minute/3Minutes/5Minutes/10Minutes/15Minutes/30Minutes/60Minutes/None	The system will be shut down if no operation after the selected time. None means the function is not effective.
Low Power	None/Only Once	The system will take which alarm scheme when the



	/Always	power of battery is going to be used up.
Info Input	Before/After/None	Set up the time when inputting archive information.
Filter Freq	[50Hz/35Hz]/[50Hz/25Hz]/[60Hz/25Hz]/[60Hz/35Hz]	Set up the parameter of AC Filter and EMG Filter.
Language	[English]/[Chinese], etc.	Set up the default system language.
K-B Sound	On/Off	When pressing the key on keyboard, the instrument will make a sound if "On" is selected. otherwise it will no sound.
Demo Mode	On/Off	The system will run under demo version, if "On" is selected. otherwise it will run under normal version.

## 12.9 Sampling Settings

Select  in the main interface, can start the following sampling setting dialogue box.

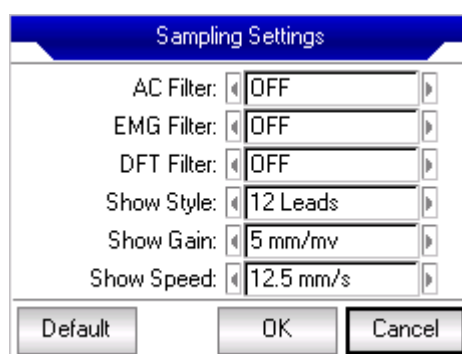


Select the button **【Default】**, the sampling settings will back to the default.


The each function of option is as following:

item	Optional content	remark
AC Filter	[ON]/[OFF]	Setting of default using AC Filter or not.
EMG Filter	[ON]/[OFF]	Setting of default using EMG Filter or not.
DFT Filter	[ON]/[OFF]	Setting of default using DFT Filter or not.
Show Style	[3 Leads]/[6 Leads]/[12 Leads]	Setting of default show style.
Show Gain	[5mm/mV]/[10mm/mV]/[20mm/mV]	Setting of default show gain.
Show Speed	[12.5mm/s]/[25mm/s]/[50mm/s]	Setting of default show speed.

## 12.10 Analysing Parameter Settings.

Select the  button in the main interface can start the following analysing parameter setting dialogue box:

The settings here will affect the diagnose hint of the real-time analysis, archive review and print report during sampling.

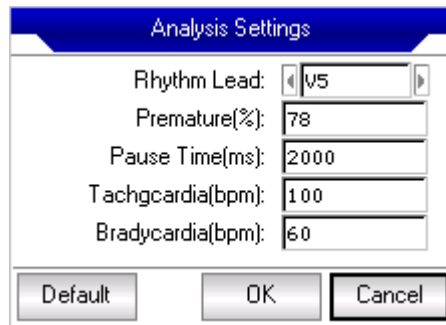


Select the button **【Default】** , the system settings will back to the default.


Refer to follow:

item	remark
Rhythm Lead	Setting the rhythm lead to analyze heart rate and print ECG in rhythm print mode.
Premature	The system will use the inputted value as a standard of judging premature beat .
Pause Time	The system will use the inputted value as a standard of judging pause beat.
Tachycardia	The system will use the inputted value as a standard of judging tachycardia.
Bradycardia	The system will use the inputted value as a standard of judging bradycardia.

### 12.11 Print Settings



Select the  button in the main interface , can start the print setting dialogue box like illustrate:

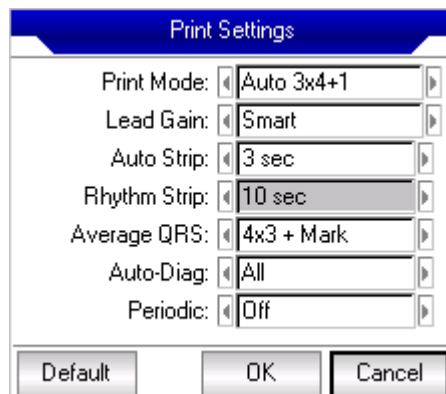

Select the button **【Default】** , the print settings will back to the default.


In this interface, the automatic mode option can only be effective when select "auto" in **【Print Mode】** .


item	Optional content	remark
Print Mode	[Auto 3×4+1] /[Auto3×4] /[Auto2×6+1] /[Auto2×6]/[Auto 4×3] /[Rhythm2]/[Rhyth m 3 ]/[Rhythm 4]/[Manual]	The selection will be used as the default print mode.
Lead Gain	Smart/Current	The selection will be used as the default Lead Gain. "Smart" means that the system will adjust the lead gain automatic to fit the height of paper while printing. "Current" means that the system will use the screen lead gain while printing.
Auto strip	3Sec/4Sec/5Sec /6Sec/8Sec/10Sec / 15Sec/20Sec/25S ec	The selection will be used as the default print time legth of strip.
Rhythm strip	10Sec/15Sec/20S ec /25Sec/30Sec	Under the print mode is "Rhythm 2", "Rhythm 3" Or "Rhythm 4", the system will use the select option as the print time length of rhythm strip.
Average QRS	[2×6]/[2×6+Mark]/[ 3×4]/[3×4+Mark]/[ 4×3]/[4×3+Mark]/[ None]	Under the print mode is "Rhythm" Or "Auto",the system will use the select option as the default print style of average QRS.
Auto-Diag	All/Data /Conclusion/None	The auto-diagnose contains 2 parts of data and conclusion, user can print one of them only as his wish.
Periodic	[per1Min]/[per 2 Min]/[per3Min]/[pe r5Min]/[per10Min]/ [per20Min]/[per30 Min]/[per 60 Min]/[off]	The system will print ECG periodically in the select time interval, If the option of print mode is selected as "Manual", the system will print in "Auto3 × 4 + 1". Otherwise, the system will print in selected mode.

Note: Auto strip, Rhythm strip, Average QRS,Auto-Diag,Periodic are available when "Auto" Or "Rhythm " print mode is selected.

## 12.12 Checking Electrodes Placement




Select  button on the main interface can start the following lead emplace illustration interface:

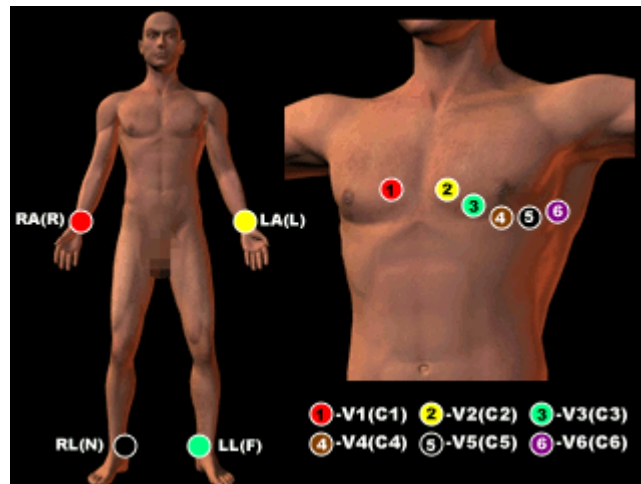

Select any key can exit this interface.

### 12.13 About Us

Select  button in the main interface can start the following information interface related to this instrument.


This interface shows the instrument name, version, company name, copyright and company contact detail.

## 13 Troubleshooting

### 13.1 Turn off automatically

- ① Please check whether the power of battery is used up. Overdischarge control circuit of the battery acts.
- ② Please check whether the alternating current voltage is too high. Overvoltage control circuit acts.
- ③ Please check whether the alternating current disturb too high, whether the fix knob of lead plug too tight. Shut automatically is for protecting circuit when overload.

### 13.2 AC interference

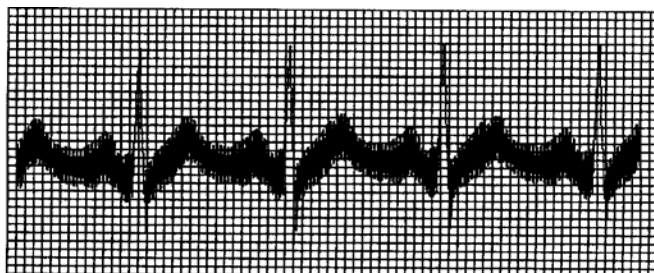

- ① Is the ECG device ground cable proper?
- ② Are the electrode and lead connected properly?
- ③ Is the electrode and skin covered with enough Gel?

- ④ Is the metal bed grounding proper?
- ⑤ Does the patient touch the wall or metal sickbed?
- ⑥ Does other people touch the patient?
- ⑦ Whether there is powerful electric device working beside ECG device? For example: X radial


device or B-Ultrasound devices.

### 13.3 EMG interference



- ① Whether the patient room is comfortable.
- ② Is the patient nervous?
- ③ Is the sickbed too narrow?

### 13.4 Baseline drift

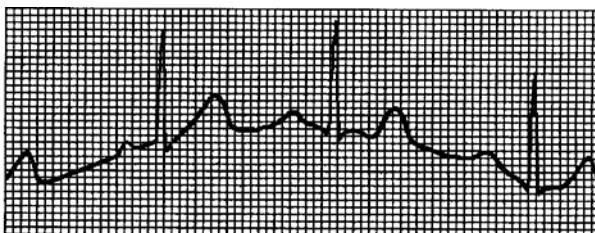

- ① Verify the electrode attachment and lead wire performance.
- ② Check the connection between patient cable and electrodes.
- ③ Check the cleaning of electrode and patient skin. Is the electrode and skin covered with enough Gel?
- ④ Keep the patient from motion or hyperventilation.
- ⑤ Is the connection between lead and electrode proper?

Please uses filter if still having above-mentioned interference.

### 13.5 Troubleshooting List

Phenomenon	Reason	Resolve method
Disturbance too big, the waveform is in disorder	<ol style="list-style-type: none"> <li>1-Whether the ground cable proper.</li> <li>2-The connection of leads is not stable.</li> <li>3-Whether there is disturbance from alternating current.</li> <li>4-Patient is nervous</li> </ol>	<ol style="list-style-type: none"> <li>1-Please check the lead, ground cable and power supply.</li> <li>2-Please dispose the patient in proper state.</li> </ol>
Baseline is rough	<ol style="list-style-type: none"> <li>1-Disturbance from alternating current is too fierce.</li> <li>2-Patient is nervous and the disturbance of EMG too strong</li> </ol>	<ol style="list-style-type: none"> <li>1-Change a comfortable environment for patient</li> <li>2-If the sickbed is metal, please change it.</li> <li>3-The power line and lead is not parallel or too close.</li> </ol>
Wave form is not regular, with too great wave or beeline	<ol style="list-style-type: none"> <li>1-The conductivity of electrode is not well.</li> <li>2-Power of battery is used up</li> <li>3-Contact between electrode and skin is not proper.</li> <li>4- The plug between lead and main unit is not tight.</li> <li>5-The contact between lead and electrode is not proper.</li> </ol>	<ol style="list-style-type: none"> <li>1-Use alcohol of high quality.</li> <li>2-Clean the electrode and patient's skin where touch the electrode</li> <li>3-Charge the battery</li> </ol>
Baseline drift	<ol style="list-style-type: none"> <li>1-Power of battery is used up</li> <li>2-Patient is moving</li> </ol>	<ol style="list-style-type: none"> <li>1-Charge the battery</li> <li>2-Keep patient hold still</li> </ol>
Waveform is not clear.	<ol style="list-style-type: none"> <li>1-The printer head is dirty</li> <li>2-The paper is not right</li> </ol>	<ol style="list-style-type: none"> <li>1-Clean the printer head with alcohol when the power is off, use the printer head after the alcohol is volatiled.</li> <li>2-Use the appointed thermal print paper.</li> </ol>

## **14 Maintenance and Storage**

### **Maintenance**

14.1 Customer is not permitted to open the instrument, in archive of any electronic shock. Any maintenance or update should execute by the trained and authorised professionals from Trimpeks. The maintenance should be done with the original accessories from Trimpeks Medical.

14.2 Please pull out the power supply plug when power off. If the device out of use for long time, please put the device in a shady cool dry place, and the device should be charged once every three months.

14.3 Under the condition of normal use according to the user manual and operation notice, if this instrument has any problem, please contact with our customer service department. Our company has the sales record and customer archives for each instrument. The customer has two year's warranty service from the beginning of shipping date according to the below time and condition. To supply all-around and fast maintenance service to our customers, please mail the maintenance card to us in time.

14.4 Our company may adopt the ways of instruction, mailing to company by courier, visiting customers' company, etc to carry out the maintenance promise.

14.5 Even in the period of free maintenance, we charge for reparation in the following archives:

14.5.1 Faults or damnification caused by misuse because not operate according to user manual and operation notice.

14.5.2 Faults or damnification caused by dropping accidentally when users move after purchasing.

14.5.3 Faults or damnification caused by preparation, reconstruction, decomposition, etc outside of our company.

14.5.4 Faults or damnification caused by natural disasters such as fire, flood, earthquake, etc.

14.5.5 Faults or damnification caused by unapt thermal recording paper.

14.6 The free maintenance period for spare parts and fray parts is half a year. Power cable, recording paper, operation manual and packing material are excluded.

14.7 Our company is not responsible for the faults of other connecting instruments cause by the faults of this device directly or indirectly.

14.8 The free maintenance service will be canceled if we find the protection label has been destroyed.

### **Storage**

- The temperature of preservation should be  $-10^{\circ}\text{C}\sim+55^{\circ}\text{C}$ , the relative humidity should less than 95%, The room for preservation should be ventilative and without causticity gas.

# firstMED

## CONTEC MEDICAL SYSTEMS CO., LTD

No.112 Qinhuang West Street, Economic & Technical  
Development Zone, 066004 Qinhuangdao, Hebei Province  
PEOPLE'S REPUBLIC OF CHINA



**Shanghai International Holding Corp. GmbH (Europe)**  
EiffestraÙe 80, 20537 Hamburg Germany

## **İthalatçı/Distributor:**

**Trimpeks İth.İhr.Tur.ve Tic.A.Ş.**

Sultan Selim Mah. Yunus Emre Cad. No:1/11

Kağıthane 34415, İSTANBUL TÜRKİYE

Tel +90 212 319 50 00 | Fax +90 212 319 50 50

Read the instructions carefully  
before using this device.



**CE**  
**0123**

Firstmed\_IB\_ECG-300\_verA\_00  
Revision Date: 10/11/2017