

## SBL SERIES Disposable Surgical Blade

### USER MANUAL

- Hands are washed and dried appropriately for the procedure to be performed.
- The packaging is checked to see if it is deformed. Do not use deformed packaging.
- The correct blade is selected for the operation.
- The product is carefully removed from the sterile package and placed on the blade/scalpel handle.
- The patient is brought to the appropriate position and the procedure is performed.
- After the process is finished, the product is thrown into the medical waste bin.

#### Attention:

Store in a clean, ventilated and non-corrosive gas environment.

Operating/Storage Conditions: Temperature:  $-10^{\circ}\text{C} - 40^{\circ}\text{C}$   
Humidity:  $\leq 80\%$



Disposable



Caution

STERILE R

Method of Sterilization Using Radiation.

#### Sterilance Medical (Suzhou) Inc.

No. 168 PuTuoShan Road New District 215153  
Suzhou, Jiangsu PEOPLE'S REPUBLIC OF CHINA



Emergo Europe B.V.

Westervoortsedijk 60, 6827 AT Arnhem, The Netherlands

**İthalatçı/Distributor: Trimpeks İth. İhr. Tur. ve Tic. A.Ş.**

Sultan Selim Mah. Yunus Emre Cad. No:1/11

Kağıthane 34415 İSTANBUL TÜRKİYE | [www.trimpeks.com](http://www.trimpeks.com)

Tel +90 212 319 50 00 | Fax +90 212 319 50 50

Made in P.R.C

CE  
0123



STERILE R